


ESC2EYESTR & EPC500ST Pendant Quick Guide

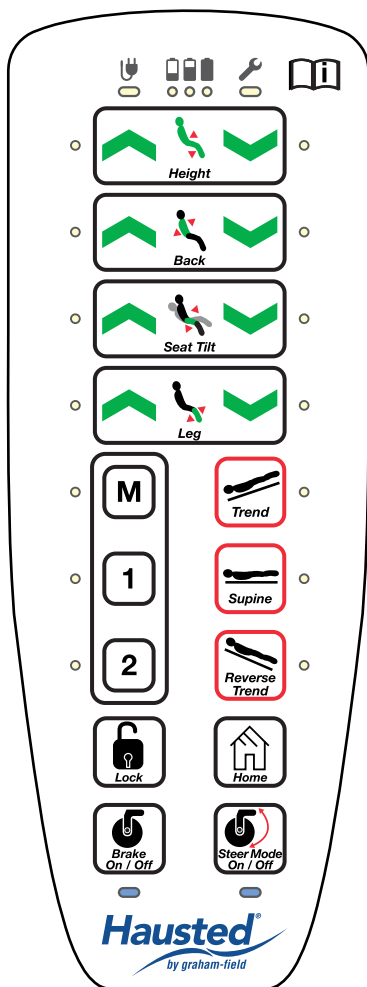

Pendant Lock

Press and hold for three seconds. All LEDs will flash yellow five times, then become solid yellow and followed by three audible beeps indicating locked.



Pendant Unlock

Press and hold BOTH buttons at the same time. All LEDs will flash yellow with three audible beeps indicating unlocked.

All LEDs will continue to flash yellow until BOTH buttons are released. Once released the LEDs will turn solid white.



Home Position

Press and hold until desired position. Chair automatically raises back section, lowers leg section, and tilts seat section simultaneously: chair also automatically adjusts height.

To Pre-Set Position 1 or 2

Step 1:

Have chair in the desired position.


Step 2:

Press and hold M button for 3 seconds. The LED will begin to flash yellow.

Holding the M button down, press 1 or 2 and hold both down until the audible beeping stops and the yellow LED on the M button turns white.

Memory is now set for the selected number.

A solid yellow LED confirms your position has been set. This LED will turn white once both buttons are released.

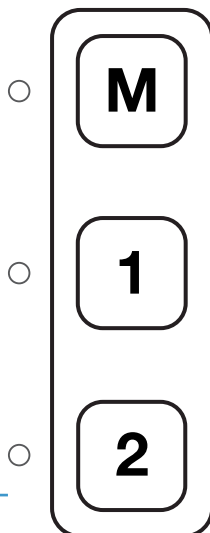

Steer Mode

Press STEER MODE button to toggle on/off. When steer mode is activated, LED is blue; when steer mode is off, LED goes out.



Brake

Press BRAKE button to toggle on/off. When brakes are locked (on), LED blue; when brakes are unlocked (off), LED goes out.

LOW Battery Indicator



Battery Level High ($\geq 75\%$):
Illuminates red, yellow, & green



Battery Level Medium ($\geq 50\%$):
Illuminates red & yellow



Battery Level Low ($< 50\%$):
Illuminates red



Battery Level Critical ($< 20\%$):
Flashing red Chair will also emit beeping sound when raising or lowering

