


# 5B800 & 5E82 Pendant Quick Guide

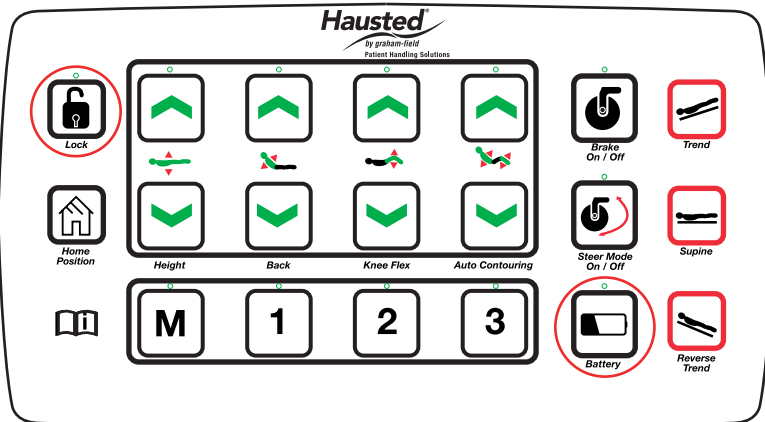


**Lock**

## Pendant Lock

Press and hold. All UP-LED's will flash five times and then become solid followed by three audible beeps indicating locked.



**Lock**

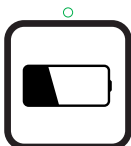

**Battery**

## Pendant Unlock

Press and hold BOTH buttons at the same time. All UP-LED's will flash with three audible beeps indicating unlocked.


All UP-LED's will continue to flash until BOTH buttons are released.



**Home  
Position**

## Home Position

Press and hold. Stretcher automatically lowers back and Knee-Flex sections and adjusts to lowest height of 22 inches.

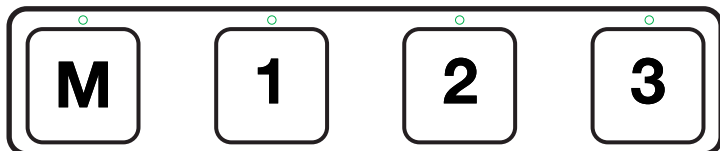

## To Pre-Set Position 1, 2, or 3 for OR, Pre-Op or Surgeon Specific

### Step 1:

Have stretcher in the desired position


### Step 2:

Press the M button and hold, green light will begin to flash.

Holding the M button down, press 1, 2, or 3 and hold both down until the audible beeping stops and the green light on the M button goes out. Memory is now set for the selected number.

A solid green light confirms your position has been set.

This light will go out once both buttons are released.

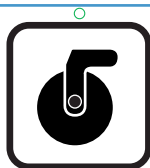

**Brake  
On / Off**

### Caster Brake

Press ONCE to lock all casters (audible beep and LED on)

Press again to unlock all casters (audible beep and LED off)

To prevent unintended movement, brakes lock automatically after stretcher is stationary for 3 consecutive minutes. Before brakes engage, a Brake alarm will sound (5 quick beeps) 5 seconds before.



**Steer Mode  
On / Off**


### Caster Steer Mode

Press ONCE to STEER stretcher (audible beep and LED on)

Press again to return to neutral (audible beep and LED off)

Note: Steer mode disabled on stretchers with Navigator Smart Drive System

Note: Engaging Steer mode through the pendant only locks the Foot-End casters into steer mode

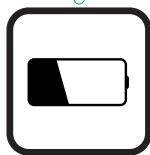

**Battery**

### LOW Battery Indicator

Illuminates steady green when battery is 20% or less.

Stretcher will emit audible beeping sound when using the unit.