


ESC2EYEST-MB & EPC500ST-MB Pendant Quick Guide

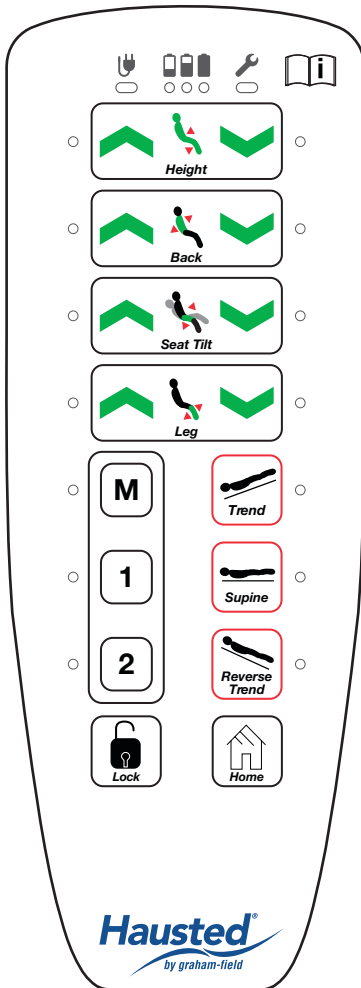

Pendant Lock

Press and hold. All LED's will flash yellow five times, then become solid yellow and followed by three audible beeps indicating locked.



Pendant Unlock

Press and hold BOTH buttons at the same time. All LED's will flash yellow with three audible beeps indicating unlocked.

All LED's will continue to flash yellow until BOTH buttons are released. Once released the LED's will turn solid white.



Home Position

Press and hold. Chair automatically raises back section, lowers leg section, tilts seat section to 5° simultaneously, and adjusts chair height to lowest position (20").



To Pre-Set Position 1 or 2

Step 1:

Have chair in the desired position


Step 2:

Press and hold M button, the LED will begin to flash yellow.

Holding the M button down, press 1 or 2 and hold both down until the audible beeping stops and the yellow LED on the M button turns white.

Memory is now set for the selected number.

A solid yellow LED confirms your position has been set. This LED will turn white once both buttons are released.

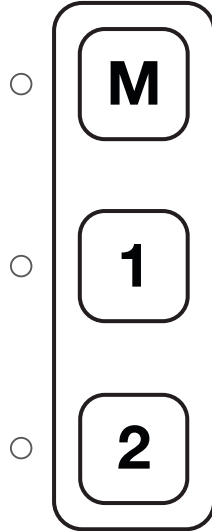

LOW Battery Indicator



Battery Level High ($\geq 75\%$):
Illuminates steady red, yellow, & green



Battery Level Medium ($\geq 50\%$):
Illuminates steady red & yellow



Battery Level Low ($< 50\%$):
Illuminates steady red



Battery Level Critical ($< 20\%$): Flashing red
Chair will also emit beeping sound when raising or lowering

