



4160DoST PEDIATRIC CARE STRETCHER

OPERATING MANUAL

SAVE THIS MANUAL FOR FUTURE USE.

CONTENTS

INTRODUCTION – A WORD FROM GF HEALTH PRODUCTS, INC.....	3
INDICATIONS FOR USE.....	3
SERVICE INFORMATION	3
ADVISORY.....	3
1 LIST OF WARNINGS AND CAUTIONS.....	3
SIGNIFICANCE OF SAFETY STATEMENTS.....	4
WARNING / CAUTION / NOTICE SUMMARY.....	4
WARNING: TO REDUCE THE RISK OF PERSONAL INJURY	4
WARNING – CAUTIONS AND PROPER OPERATION	4
2 UNCRATING INSTRUCTIONS	5
IMPORTANT – REPORT ANY SHIPPING DAMAGE IMMEDIATELY:	5
NOTICE – POSSIBLE EQUIPMENT DAMAGE:	5
WARNING – PERSONAL INJURY HAZARD:.....	5
UNPACKING INSTRUCTIONS	5
3 OPERATING INSTRUCTIONS.....	6
3.1 BRAKING AND STEERING OPERATION.....	6
3.1.1 APPLYING THE BRAKES	6
3.1.2 RELEASING THE BRAKES	6
3.2 LITTER TOP HEIGHT ADJUSTMENT	7
3.2.1 HEIGHT ADJUSTMENT.....	7
3.2.2 LOWERING LITTER TOP	7
3.2.3 TRENDELENBURG ADJUSTMENT.....	7
3.2.4 REVERSE TRENDELENBURG ADJUSTMENT.....	7
3.3 FOWLER BACKREST OPERATION	8
3.3.1 RAISING THE BACKREST	8
3.3.2 LOWERING THE BACKREST	8
3.4 RAIL OPERATION	9
3.4.1 RAISING SIDE RAIL TO FULL HEIGHT.....	9
3.4.2 RAISING SIDE RAIL TO HALF HEIGHT	9
3.4.3 LOWERING SIDE RAIL.....	9
4 SPECIFICATIONS	10
ENVIRONMENTAL CONDITIONS.....	10
MAXIMUM WEIGHT CAPACITY	10
5 PREVENTIVE MAINTENANCE FOR THE USER	11
6 OPTIONAL ACCESSORIES.....	12
7 GF HEALTH PRODUCTS, INC. LIMITED WARRANTY.....	13
8 DISPOSAL AND KEY TO SYMBOLS.....	14
DISPOSAL.....	14
KEY TO SYMBOLS.....	14
9 INDEX.....	15

COPYING PROHIBITED

This manual is protected by Federal Copyright Law, which provides for damages of up to USD \$20,000, as well as criminal fines and imprisonment, for unauthorized copying.

© 2016 GF Health Products, Inc. All Rights Reserved Printed in U.S.A.



INTRODUCTION — A WORD FROM GF HEALTH PRODUCTS, INC.

This manual contains important information on proper use and maintenance of the Hausted® Pediatric Care Stretcher. All personnel involved in the use and maintenance of this equipment must carefully review and comply with the warnings, cautions and instructions contained in this manual. These instructions are important to protect the health and safety of personnel operating a Hausted Pediatric Care Stretcher model 4160DoST and should be retained in a conveniently accessible area for quick reference.

Complete instructions for uncrating and putting your new equipment in service, as well as equipment drawings, have been furnished. If missing, contact GF Health Products, Inc. ("Graham-Field") for replacement copies, giving the serial number and model number of the unit.

GF Health Products, Inc. carries a complete line of accessories for use with these stretchers; your representative will gladly review these with you.

INDICATIONS FOR USE

The Hausted Pediatric Care Stretcher model 4160DoST is intended for pediatric-age patient transport, treatment, and recovery.

SERVICE INFORMATION

A thorough preventive maintenance program is essential to safe and proper unit operation. This manual contains maintenance schedules and procedures which should be followed for satisfactory equipment performance.

We encourage you to contact GF Health Products, Inc. with maintenance concerns.

ADVISORY

A listing of the safety precautions to be observed when operating and servicing this equipment can be found in Section 1 of this manual. DO NOT operate or service the equipment until you have become familiar with this information. ANY alteration of this equipment NOT AUTHORIZED or performed by GF Health Products, Inc., could affect its operation, will VOID the warranty, could violate national, state, and local regulations, and could jeopardize your insurance coverage.

Info: *Column 1 below applies only if product was purchased outside the U.S.*

	EU REP
	EU Authorized Representative: AR Experts BV Boeingavenue 209 1119 PD Schiphol-Rijk The Netherlands www.ar-experts.eu

	Manufactured by: GF Health Products, Inc. 1 Graham-Field Way Atlanta GA 30340-3140 1.770.368.4700 Main 1.770.368.2386 Fax www.grahamfield.com www.Hausted.com
---	--

Info: *The base language of this document is ENGLISH. Any translations must be made from the base language document.*

1 LIST OF WARNINGS AND CAUTIONS

⚠ IMPORTANT: Before using the Hausted Pediatric Care Stretcher, please read and adhere to the following safety precautions and warnings. Failure to do so could result in serious personal injury or damage to the Stretcher.

Always consult your healthcare professional to determine safe methods most suitable for your individual abilities. Protect yourself, your attendant, and the Hausted Pediatric Care Stretcher by having it serviced regularly. If you experience any malfunction, contact your Graham-Field authorized distributor immediately, as a hazardous condition could result, causing personal injury or damage to the Stretcher.

Periodic inspection, adjustment and replacement of worn parts are necessary to provide years of excellent service. Maintenance **MUST** be performed by qualified personnel **ONLY**.

SAVE THESE INSTRUCTIONS.

SIGNIFICANCE OF SAFETY STATEMENTS

Please note the following special statements, used throughout this manual, and their significance:

- ⚠ **DANGER:** Indicates a potential hazard situation or unsafe practice that, if not avoided, will result in death or serious personal injury.
- ⚠ **WARNING:** Indicates a potential hazard situation or unsafe practice that, if not avoided, could result in death or serious personal injury.
- ⚠ **CAUTION:** Indicates a potential hazard situation or unsafe practice that, if not avoided, could result in minor or moderate personal injury.
- ▲ **NOTICE:** Indicates a potential hazard situation or unsafe practice that, if not avoided, could result in product or property damage.

Info: Provides application recommendations or other useful information to ensure that you get the most from your product.

WARNING / CAUTION / NOTICE Summary

The following is a listing of the safety precautions which must be observed when operating and servicing this equipment. These precautions are repeated (in whole or in part), where applicable, throughout the manual.

WARNING: To Reduce the Risk of Personal Injury

- ⚠ **WARNING: LACERATION HAZARD** — When cutting bands always use a tool specifically designed for that purpose. This will help to avoid personal injuries frequently incurred when bands are cut and tension released.
- ⚠ **WARNING: DO NOT** sit on end — tipping may occur.
- ⚠ **WARNING:** Ensure IV rod is inserted completely into socket up to the arrow before applying load.

WARNING — CAUTIONS AND PROPER OPERATION

- ⚠ **WARNING:** Equipment is **NOT SUITABLE** for use in the presence of flammable anesthetic mixture with air, or oxygen, or nitrous oxide.
- ⚠ **WARNING:** Pediatric Care Stretcher model 4160D0ST has a maximum weight capacity of 325 lb (175 kg) **EVENLY DISTRIBUTED**.
- ⚠ **WARNING:** The patient transport position is pushing from the patient head end in the supine position.
- ⚠ **WARNING:** Patient entry, egress, and transfer should **ALWAYS** be done with the brakes locked.
- ⚠ **WARNING:** The caster braking system should **ALWAYS** be locked when patient is **NOT** in transport.
- ⚠ **WARNING:** The caster braking system should **ALWAYS** be locked and patient side rails in upright locked position when patient is left unattended.
- ⚠ **WARNING:** **ALWAYS** ensure side rails are in upright locked position **BEFORE** leaving patient unattended.
- ⚠ **WARNING:** Patient entry, egress, and transfer from the stretcher should always be from the center side rail location with the side rail in the down position and brakes locked.
- ⚠ **WARNING:** Steam cleaning and pressure washing of stretcher is not recommended and can void warranty.
- ⚠ **WARNING:** **DO NOT** modify the equipment without the authorization of the manufacturer.
- ⚠ **CAUTION:** When lowering the rails, ensure patient and caregiver body and extremities are clear of pinch points before operating the rail.
- ⚠ **WARNING:** Keep hands clear of pinch points.
- ⚠ **WARNING:** Inform shipper of any damage — leave carton intact. Leave equipment in the receiving area until inspection is complete.
- ▲ **NOTICE:** The crate contains fragile, expensive medical equipment. Uncrate and handle carefully. If after uncrating the equipment you find any damage (no matter how slight), report the damage to GF Health Products, Inc.
- ⚠ **WARNING:** When cutting bands, always use tool specifically designed for that purpose. This will help avoid personal injuries possibly incurred when bands are cut and tension is released.

2 UNCRATING INSTRUCTIONS

IMPORTANT — REPORT ANY SHIPPING DAMAGE IMMEDIATELY:

⚠ WARNING: Inform shipper of ANY damage — leave carton intact. Leave equipment in the receiving area until inspection is complete.

NOTICE — POSSIBLE EQUIPMENT DAMAGE:

▲ NOTICE: The crate contains fragile, expensive medical equipment. Uncrate and handle carefully. If after uncrating the equipment you find ANY damage (no matter how slight), report the damage to GF Health Products, Inc.

WARNING — PERSONAL INJURY HAZARD:

⚠ WARNING: When cutting bands, ALWAYS use tool specifically designed for that purpose. This will help avoid personal injuries possibly incurred when bands are cut and tension is released.

UNPACKING INSTRUCTIONS

IMPORTANT: Follow each step in the order shown in these instructions.

Your Hausted equipment has been carefully packed at our manufacturing plant to ensure safe shipment to your medical facility. There are several procedures you must follow to put your new equipment in service. These procedures only take a few minutes to complete and all are required to ensure proper operation of the equipment.

1. Cut the two bands around the shipping carton.
2. Remove the top half of the carton and cut one side of the bottom half.
3. Remove the equipment from the carton.
4. Check to see if all features of the equipment work properly. If all the features work, advance to step 5. If any features do not work properly, call GF Health Products, Inc. at 1.770.368.4700.
5. Clean the equipment using mild detergent to remove any dirt accumulated during shipment, and place the equipment into service.

3 OPERATING INSTRUCTIONS

3.1 BRAKING AND STEERING OPERATION

⚠ WARNING—PERSONAL INJURY HAZARD AND / OR EQUIPMENT DAMAGE HAZARD:

The caster braking system should always be locked when patient is not in transport.

The caster braking system should always be locked and patient side rails in upright locked position when patient is left unattended.

3.1.1 Applying the Brakes

Activate the four-wheel central braking system by depressing the red pedal at any of the unit's four corners (Figure 3-1). To fully engage brakes, press pedal to approximately 45°. All four caster wheels will then be locked from swiveling and rotating.

3.1.2 Releasing the Brakes

Depress the green pedal at any of the unit's four corners until the pedal is in a horizontal position (Figure 3-2). All four wheels will then rotate and swivel freely.

⚠ WARNING: The caster braking system should always be locked when patient is NOT in transport.

3.1.3 Applying the Steering Lock

Depress the green pedal at any of the unit's four corners downward into locked position (Figure 3-3). push the stretcher forward. One caster at the foot end will lock into a non-swivel mode. This allows straight steering by the attendant, when pushing from the head end.

3.1.4 Releasing the Steering Lock

Depress the red pedal at any of the four corners of the unit, until the pedal is in a horizontal position (Figure 3-2). all four wheels should rotate and swivel freely.



Figure 3-1

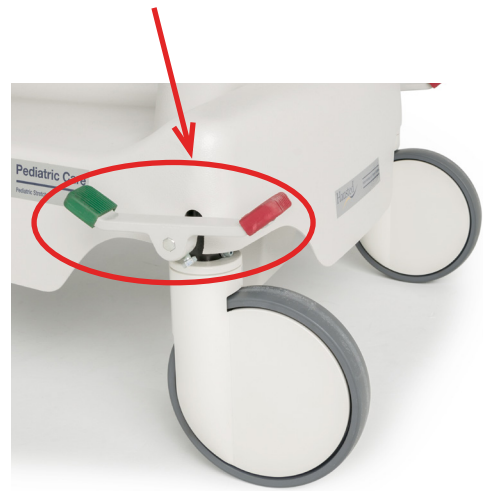


Figure 3-2

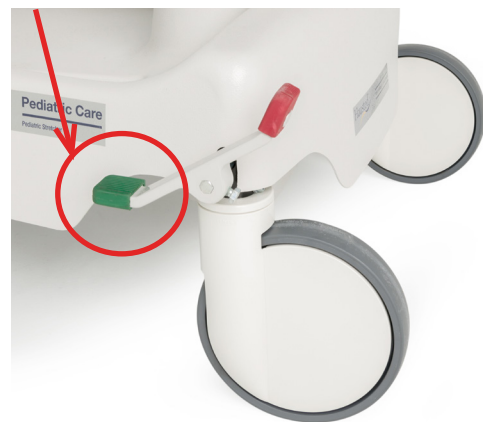


Figure 3-3

3.2 LITTER TOP HEIGHT ADJUSTMENT

3.2.1 Height Adjustment

Press pump pedal to floor (Figure 3-4), then release. Repeat this process until desired height is obtained. Use smooth strokes on pump pedal to ensure patient comfort.



Figure 3-4

3.2.2 Lowering Litter Top

Press down on two release pedals at the same time (Figure 3-5) until desired height is obtained.

▲ **NOTICE: DO NOT stand on release pedals.**

3.2.3 Trendelenburg Adjustment

Place unit at maximum height: see “Height Adjustment” above. Press down on the release pedal nearest the head end (Figure 3-6) until desired position is obtained, then remove pressure.



Figure 3-5

3.2.4 Reverse Trendelenburg Adjustment

Place the unit at maximum height: see “Height Adjustment” above. Press down on release pedal nearest foot end (Figure 3-7) until desired position is obtained, then remove pressure.



Figure 3-6



Figure 3-7

3.3 FOWLER BACKREST OPERATION

3.3.1 Raising the Fowler Backrest

Grasp the fowler frame tube (Figure 3-8) and lift up on the fowler. when desired height is reached, lower the fowler slightly until the ratchet plates are engaged. Be sure the fowler is locked into place by pushing down slightly on the fowler before releasing the tube.

Info: *The amount of manual lifting required will vary depending on patient size; the gas spring is intended only to assist in lifting patient.*

3.3.2 Lowering the Fowler Backrest

Grasp the fowler frame tube with hand and grasp the ratchet release handle with finger tips. Squeeze the ratchet release handle and the fowler frame tube together while lifting slightly on the fowler. Once the ratchet supports are disengaged, while lowering fowler to desired position, continue squeezing ratchet release handle. Be sure the fowler is locked into place by pushing down slightly on the fowler before releasing the tube.

Info: *The amount of manual support required will vary depending on patient size; the gas spring is intended only to assist in lowering patient.*



Figure 3-8

3.4 RAIL OPERATION

⚠️ WARNING — PROPER OPERATION HAZARD:

Patient entry, egress and transfer should ALWAYS be done with caster braking system locked.

Caster braking system should ALWAYS be locked and patient side rails in upright locked position when patient is left unattended.

3.4.1 Raising the Rail

Grasp the rail top tube in the center of the rail (Figure 3-9a).

With your free hand, squeeze the rail trigger (Figure 3-9B) and lift the rail all the way up (Figure 3-9C).

Once the rail is all the way up, release the rail trigger and lower the rail slowly until the trigger engages.

Check rail lock by trying to squeeze the rail trigger.

If you cannot squeeze the trigger, the rail is locked securely.

Release the rail top tube.

3.4.2 Raising the Rail to Half-Height

Grasp the rail top tube in the center of the rail (Figure 3-9a) and raise the rail until it engages.

Check rail lock by trying to squeeze the rail trigger.

If trigger will not squeeze, the rail is locked securely.

Release rail top tube.

3.4.3 Lowering the Rail

Grasp the rail top tube in the center of the rail.

With your free hand, squeeze the rail trigger (Figure 3-9C) while lifting slightly on the rail. This should allow the rail trigger to release the trigger lock.

Lower the rail slowly (Figure 3-10).

If "half-height" is desired, release the rail trigger as soon as the rail starts lowering.

If "full-down" is desired, continue to squeeze the trigger until the rail is all the way down.

⚠️ **CAUTION: When lowering the rails, ensure patient and caregiver (or attendant) body and extremities are clear of pinch points before operating the rail.**

⚠️ **WARNING: ALWAYS ensure side rails are in upright locked position BEFORE leaving patient unattended.**



Figure 3-18A

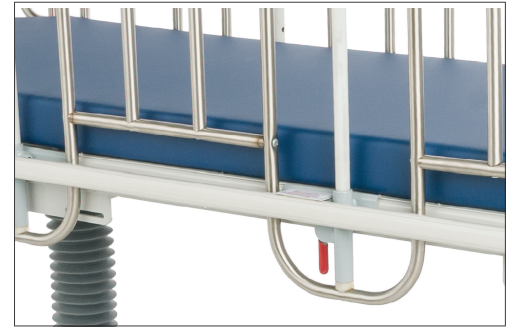


Figure 3-18B



Figure 3-18C

4 SPECIFICATIONS

ENVIRONMENTAL CONDITIONS

Operating Conditions	
Temperature	10°C to 40°C
Relative Humidity	20% to 90% @ 30°C
Atmospheric Pressure	700 hPa to 1060 hPa

Storage and Transport Conditions	
Temperature	-10°C to 50°C
Relative Humidity	20% to 90% @ 30°C
Atmospheric Pressure	700 hPa to 1060 hPa

MAXIMUM WEIGHT CAPACITY

⚠ **WARNING: Pediatric Care Stretcher model 4160D0ST has a maximum weight capacity of 325 lb (175 kg) EVENLY DISTRIBUTED.**

All Pediatric Care Stretchers have a warning label at head or foot end of stretcher top frame that states maximum weight capacity. Those models whose weight capacity has been increased can be identified by this weight capacity label at head or foot of stretcher top frame, as well as the navy bumper channel and rail cap; earlier models with lower weight capacity, and those with unchanged weight capacity, have a gray bumper channel and rail cap.

SPECIFICATIONS: PEDIATRIC CARE STRETCHER

Specification	4160D0ST
Maximum Height	37 in (94 cm)
Minimum Height	28.5 in (92.4 cm)
Overall Width	30 in (76 cm)
Overall Length	83 in (210 cm)
Patient Surface Width	25 in (63.5 cm)
Patient Surface Length	76 in (193 cm)
Backrest Degree of Movement	0° - 90°
Knee Flex Degree of Movement	-
Trendelenburg / Reverse Trendelenburg	12°
Vertical Rail Height	16 in (41 cm)
Vertical Rail Length	73 in (185 cm)
End Rail Height	16 in (41 cm)
End Rail Length	23 in (58 cm)
Spindle Gap Maximum	2.5 in (6 cm)
Spindle Gap Minimum	4 in (10 cm)
Casters	8 in Tente® (20 cm)
Maximum Weight Capacity (EVENLY DISTRIBUTED)	325 lb (175 kg)
Pad	3 in (8 cm)

5 PREVENTIVE MAINTENANCE FOR THE USER

Component	Cleaning Procedure	Schedule	Cleaning Agent *	Special Notes
Pads /Mattresses	Wipe with damp cloth to remove any foreign material	After each use	Routine hospital grade disinfectants, soap and water	Use only medium strength cleaners Do not steam clean or pressure wash
Stretcher	Wipe with damp cloth to remove any foreign material		Routine hospital grade disinfectants, soap and water	Lubricate pivot points after cleaning
Mechanical stretcher components	Wipe with damp cloth to remove any foreign material		Routine hospital grade disinfectants, soap and water	Lubricate pivot points after cleaning
Mechanical accessories	Wipe with damp cloth to remove any foreign material		Routine hospital grade disinfectants, soap and water	Lubricate pivot points after cleaning
Procedure	Schedule	Material		
Lubricate all moving and sliding parts and hinge points	Every 3 months	Lubricating oil, light-duty grease, wax stick lubricant or Never-Seez lubricant		
⚠ NEVER LUBRICATE GAS SPRING, MOTOR, OR MECH-LOCK SHAFTS ⚠				
Inspect all fasteners to ensure proper fit, position and tightness, including nuts, bolts, etc.	Every 3 months	Proper size wrench and screwdriver		
Inspect all surfaces and remove any sharp or burred areas; apply touch-up paint where required		Metal file, proper color paint (specify color when ordering)		
* Disinfecting and Cleaning Upholstery - ALWAYS follow manufacturer's recommended dilution				
Disinfectants for vinyl products	Phenolic disinfectants are the best choice for vinyl			
	Properly diluted quaternaries are also acceptable for vinyl			
	Quaternary / Isopropyl disinfectants ARE NOT recommended for vinyl			
Disinfectants for urethane products	Quaternary disinfectants are recommended for urethane			
	Quaternary / Isopropyl disinfectants are recommended for urethane			
	Phenolics SHOULD BE AVOIDED on urethane.			
Disinfectants for all products	All fabrics may be cleaned with a 1:10 dilution of household bleaches containing 5.25% sodium hypochlorite as recommended by the Centers for Disease Control in Atlanta, Georgia; there is no harmful effect on the fabric			
	Disinfectants applied at full concentration or in highly concentrated solutions will decrease the useful life of fabric			
	Iodophor-type disinfectants used on fabric may result in staining			
Soils or Stains	Use neutral soapsuds and lukewarm water; DO NOT use harsh cleansers, solvents or detergents			
Hard-To-Clean Spots	Use standard household / vinyl cleansers and a soft bristle brush on troublesome spots or stains; presoak heavy, dried-on soil			
Laundering	Laundering Vinyl-laminated, Polyurethane-coated, or Rubber-coated fabric IS NOT recommended; laundering may substantially decrease the useful life of the fabric			

▲ NOTICE – POSSIBLE EQUIPMENT DAMAGE HAZARD: Steam cleaning and pressure washing of stretcher is not recommended and can void warranty.

Info: For more detailed information, please contact GF Health Products, Inc. at 1.770.368.4700.

Info: In addition to the User Preventive Maintenance, a more detailed Preventive Maintenance Program is also required to keep the equipment in good working order. This Preventive Maintenance Program is found on our website, at www.Hausted.com.

6 OPTIONAL ACCESSORIES

Item No.	Description
H00011P00	1" Thick Foam Padded Side and End Rails, Color: Mauve
H00001200	Safety Straps with Buckle
HAPC01200	Safety Straps with Hook and Loop
H000E1700	IV rod 42" Stainless Steel
H00001800	Telescoping IV rod, 27" to 60" 2 Section, with Holder, Stainless
H12674000	Infusion Pump IV pole 1-1/8" Diameter, 42" Long, Removable
H12845000	Vertical Oxygen Tank Holder
HIVSTOW00	Mobile IV Stand Attachment
HIVSTOW00	Fifth Wheel Assembly

⚠ WARNING: It is recommended that only accessories approved by GF Health Products, Inc. be used with this device.

Info: To order accessories, or for more detailed information on accessories, please contact GF Health Products, Inc. at 1.770.368.4700.

7 GF HEALTH PRODUCTS, INC. LIMITED WARRANTY FOR HAUSTED BRAND SCOPE OF WARRANTY

GF Health Products, Inc. ("GF") warrants to the original purchaser only that it will replace or repair components, at GF's sole discretion, that are defective in material or workmanship under normal use and service. All warranties are conditioned upon the proper use and maintenance of the products strictly in accordance with good commercial practice and applicable GF instructions and manuals. To the extent that a component is warranted by a third party, GF conveys all of its rights under that warranty to the original purchaser, to the extent permitted. This limited warranty shall only apply to defects that are reported in accordance with the provisions set forth in this warranty document within the applicable warranty period and which, upon examination by GF or its authorized representative, prove to be a warranty item. This limited warranty is not transferable.

The warranted components and time period are set forth below:

COMPONENT	PARTS WARRANTY LABOR WARRANTY*	
Frame.....	5 years.....	1 year
Casters.....	1 year.....	1 year
Hydraulics.....	1 year.....	1 year
Optional accessories.....	1 year.....	1 year
Patient weighing system.....	1 year.....	1 year
Pneumatic gas springs.....	1 year.....	1 year
Upholstered pads.....	1 year.....	1 year
Replacement parts**.....	90 days	

* Labor is not included in the warranty.

† Upholstery is only warranted on material supplied by GF.

‡ The warranty period is as designated above. If a part is replaced under warranty, the original warranty period will not be affected. All other replacement parts will be subject to the warranty period specified.

The applicable warranty period shall commence from date of shipment to the original customer, unless there is an expiration date on the component in which case the warranty shall expire on the earlier of warranty period or the expiration date.

OBTAINING WARRANTY SERVICE

Customers located in the United States who wish to report a warranty issue, must contact GF directly by calling 1.770.368.4700 or by e-mailing a request to cs@grahamfield.com. Customers located outside the United States must contact the Distributor from whom they purchased the products. In both cases, further directions will be provided once the initial contact is made. This limited warranty shall only apply to defects that are reported within the applicable warranty period. Failure to abide by the specific directions will result in denial of the warranty claim.

The warranty does not cover and GF shall not be liable for the following:

1. Defects, damage, or other conditions caused, in whole or in part, by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner;
2. Products which are not installed, used, or properly cleaned and maintained as required in the official manual for the applicable product;
3. Products considered to be of a non-durable nature including, but not limited to: filters, fuses, gaskets, lubricants, and charts;
4. Accessories or parts not provided by GF;
5. Matching of color, grain or texture except to commercially acceptable standards;
6. Changes in color caused by natural or artificial light;
7. Charges by anyone for adjustments, repairs, replacement parts, installation or other work performed upon or in connection with such products which are not expressly authorized in writing, in advance, by GF;
8. Any labor or shipping charges incurred in the replacement part installation or repair;
9. Costs and expenses of regular maintenance and cleaning; and
10. Representations and warranties made by any person or entity other than GF.

ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER

THIS WARRANTY IS GF'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. GF MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. IF ANY MODEL OR SAMPLE WAS SHOWN TO THE CUSTOMER, SUCH MODEL OR SAMPLE WAS USED MERELY TO ILLUSTRATE THE GENERAL TYPE AND QUALITY OF THE PRODUCT AND NOT TO REPRESENT THAT THE PRODUCT WOULD NECESSARILY CONFORM TO THE MODEL OR SAMPLE IN ALL RESPECTS. THIS WARRANTY IS LIMITED TO THE REPAIR OR REPLACEMENT OF THE DEFECTIVE PARTS. GF SHALL NOT BE LIABLE FOR AND HEREBY DISCLAIMS ANY DIRECT, SPECIAL, INDIRECT, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES, INCLUDING, BUT NOT LIMITED TO: DAMAGES FOR LOSS OF PROFITS OR INCOME, LOSS OF USE, DOWNTIME, COVER, OR EMPLOYEE OR INDEPENDENT CONTRACTOR WAGES, PAYMENTS AND BENEFITS. CERTAIN STATES MAY CONFER ADDITIONAL RIGHTS REGARDING WARRANTIES AND IN THOSE STATES GF'S LIABILITY AND THE LIABILITY OF GF'S SUPPLIERS, SHALL BE LIMITED TO THE FULLEST EXTENT PERMITTED BY LAW.

The warranties contained herein, together with GF's current Terms and Conditions, contain all the representations and warranties with respect to the subject matter of this document, and supersede all prior negotiations, agreements and understandings with respect thereto. The recipient of this document hereby acknowledges and represents that it has not relied on any representation, assertion, guarantee, warranty, collateral contract or other assurance, except those set out in this document.

For additional information on this product or this warranty, please contact a GF Customer Service Representative.

NOTES:

1. Additional terms and conditions may apply. See GF's General Terms and Conditions on its website: www.grahamfield.com.
2. Freight claims must be notated on the appropriate shipping documents and must be made with immediacy. International, federal and state regulations govern specific requirements for freight claims. Failure to abide by those regulations may result in a denial of the freight claim. GF will assist you in filing the freight claim.
3. Claims for any short shipment must be made within three (3) days of the invoice date.

8 DISPOSAL AND KEY TO SYMBOLS

DISPOSAL

Follow local governing ordinances and recycling plans regarding disposal of the device or components normally used in operation. The device does not generate waste or residue in operation. Any accessories not part of the device **MUST** be handled in accordance with the individual product marking for disposal.

















We recommend disassembling and dividing the equipment and components into different waste groups such as: metal, recoverable resource, and plastic for recycling or combustion.

Most plastic components are provided with a plastic types code and fiber content to aid sorting of plastic parts.

Product	Metal Scrap	Cable Scrap	Electronic Scrap	Plastic Recycling or Combustion
4160D0ST	X	X	None	X

KEY TO SYMBOLS

The following symbols are used on Hausted product labels.

	Protective Earth		Manufacturer
	Earth Ground		Keep Dry
	General Warning Sign		Fragile, Handle with Care
	CE Mark		Electrical and Electronic Equipment
	ETL		Consult Instructions for Use
	European Authorized Representative		Caution
	Disconnect before Service		Pinch Point
	Medical Device		Unique Device Identifier

www.hausted.com / www.grahamfield.com

9 INDEX

A

Accessories, optional, list 12
Adjustment, height 7
Adjustment, liter 7
Adjustment, lower 7
Advisory 3
A word from GF Health Products, Inc. 3

B

Backrest, lowering 8
Backrest, raising 8
Base language of this document 3
Brakes, applying 6
Brakes, releasing 6
Braking and steering operation 6

C

CAUTION statement, significance 3

D

DANGER / WARNING / CAUTION / NOTICE
Summary 3
Disposal 14

E

EU Authorized Representative: 2
Environmental conditions 10

F

Fowler backrest operation 8

I

Indications for use 3
Info statement, significance 4

L

Limited warranty 13
Litter top height adjustment 7

M

Manufactured by: 4
Maintenance, Preventive: 11

N

NOTICE statement, significance 3

O

Operating instructions 6

R

Rail, half-height 9
Rail, lowering 9
Rail operation 9
Rail, raising 9

S

Safety statements, significance of 3
Service information 2
Specifications: 10
Symbol key, Hausted product labels 14

T

Trendelenburg adjustment 7
Trendelenburg, reverse, adjustment 7

U

Uncrating instructions 5
Unpacking instructions 5

W

WARNINGS and CAUTIONS, list 2
WARNING statement, significance 2
Warranty, limited 13
Weight capacity, maximum 10

Hausted[®]
by graham-field



Manufactured By:
GF Health Products, Inc.
1 Graham-Field Way, Atlanta, GA 30340
Made in USA

© 2016, GF Health Products, Inc. All Rights Reserved.
Hausted is a registered trademark of GF Health Products, Inc.

HP_4160-GFH1300085-INS-LAB-RevC25



+1 770.368.4700

Information contained herein is subject to change without notice.
The most current and complete product information can be found on our website.
www.hausted.com | www.grahamfield.com

